

Storing the Carrier

- When not in use, the platform can be put in storage position by pulling the pull handle and lifting the platform to a vertical position. Insert the snap hook into the loop on the platform to lock it into the storage position.

Safety

CAUTION: Do not operate our carriers without being satisfactorily instructed in the proper operation of the carrier by the dealer or without reading the operation manual thoroughly.

- Our wheelchair carriers have been designed and engineered for years of trouble free performance. Although with everyday use some parts may become loose or worn. Regularly inspect your carrier for any worn, loose, or damaged parts. If you find anything upon inspection, **DO NOT USE THE CARRIER!** Contact your local dealer or WheelChair Carrier for any repairs that need to be made. **Failure to act may cause severe injury!**
- Our wheelchair carriers should only be used for the loading and unloading of wheelchairs for which it is designed. **DO NOT** add to or modify any part of the carrier system without first contacting the manufacturer of the carrier. Any modifications may void any warranties as well as compromise the structural integrity of the carrier.

Maintenance

Our wheelchair carriers have been designed to be as trouble free as possible for the owner. As with any mechanical device however, maintenance should be performed regularly while owning and operating our carriers.

- *WheelChair Carrier* recommends that dealers schedule a preventative maintenance inspection at least once a year on the carrier frame, the tilting cable, and all moving parts of the carrier.
- Regularly check and tighten all nuts and bolts on the carrier.
- Regularly check for paint chips and touch up any bare metal with a good gloss black enamel or lacquer to inhibit rust. WheelChair Carrier recommends checking your carrier frequently if it is subjected to salt air or road salt for any period of time.