

Introduction

Congratulations on your new *WheelChair Carrier* product purchase! The Model 010 Universal Scooter & Power Wheelchair Carrier is one of the easiest ways to transport your folding wheelchair. This manual has been written to be used as an installation guide as well as an operation guide for the end user.

Both installers and operators should read all of the available manuals thoroughly before attempting any installation, operation or maintenance of the Model 010 Universal Scooter & Power Wheelchair Carrier.

If you have any questions or comments concerning the installation or operation of this carrier, please contact your local dealer or WheelChair Carrier for additional technical information.



Caution: Be sure that the wheelchair or scooter meets the size constraints set for this product. Never exceed the posted weight capacity of the carrier.

Unpacking the Carrier

Detailed below is the inventory of parts included with your carrier. If any parts are missing or if any shipping damage is noted, immediately contact your dealer or WheelChair Carrier if purchased direct. **DO NOT** attempt to install or use the lift with any missing or damaged parts.

The Model 010 Universal Scooter & Power Wheelchair Carrier Box Contents:

- 1- Platform
- 1 - Hitch Body/Crank Body Assembly
- 1 - Short Pull Handle
- 1 - Long Pull Handle
- 4 - 1" Cap Plugs
- 4 - 1 1/4" Cap Plugs
- 2 - Small Hinge Pins
- 4 - 3/16" X 1 1/4" Roll Pins
- 3 - Ratchet Tie Down Straps
- 1 - Owner's Manual

Installation Instructions



Important!!

A minimum Class II or Class III hitch (heavy duty) is required . Air shocks may also be needed. DO **NOT** exceed 175 pound weight capacity!

CAUTION: Do **NOT** attempt to ride scooter or chair onto carrier to load or unload. Periodic tightening and lubrication of carrier parts extend life and ease operation.

Assembly

- 1.) Lay platform upside down on ground.
- 2.) Place platform hitch and support assembly upside down on platform. Be sure to line up both sets of pivot bushings. Also, be sure that the pull handle is inserted into the hole of pivot plate on platform hitch and support assembly.
- 3.) Insert both 1" X 3 1/4" pivot pins into bushings. Now drive the 4 roll pins provided into the hole in the pivot pins, locking them into place.

Mounting

- 1.) Insert the 2" X 2" square hitch attachment into the Class 3 hitch receiver **OR** insert 1 1/4" X 1 1/4" hitch attachment into the Class 2 adapter.
- 2.) Place hitch pin into hole and secure.