

Operation Instructions

- Carefully lift the folded wheelchair onto the carrier. The front wheels should rest on the Front Wheel Pan and the rear wheels should rest firmly in the Rear Wheel Cradle.

Securing the Chair to the Carrier

- Raise the Extension Post high enough that the Lock Down Arm will fit over the rear wheel. Extend the Lock Down Arm so the wheel slot fits over the rear wheel. Tighten the upper handle in order to secure the Lock Down Arm in place.



CAUTION: Do Not Overtighten Plastic Handles.

- While pushing downward on the Lock Down Arm tighten the lower handle on the Extension Post to secure the rear wheel.



CAUTION: Do Not Overtighten Plastic Handles.

- Position the rubber bungee cord through the front wheels and hook it on each side of the Front Wheel Pan. There are four slots on each side of the Front Wheel Pan for mounting the bungee cord.
- Always verify that the wheelchair is secure before operating the vehicle.