



Model #022 Tilt 'N' Tote Cover Installation Instructions

**** Be sure Locking Arm is disconnected
before installing cover**

1. Place Large Flap of cover over push handles and back of chair.

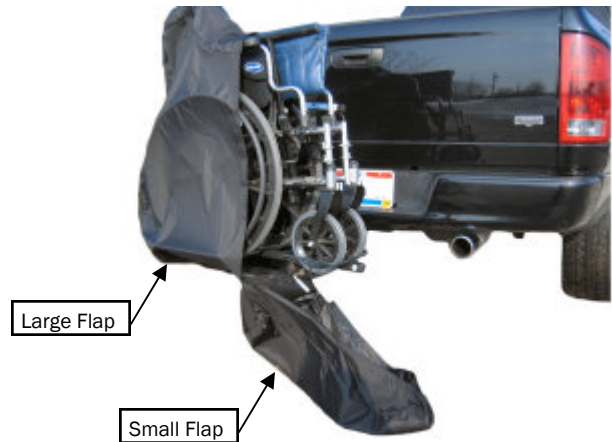

2. Pull Back Flap up. Position Locking Arm over large wheel and into the lower locking arm of carrier. Press flap closed over head of locking arm.

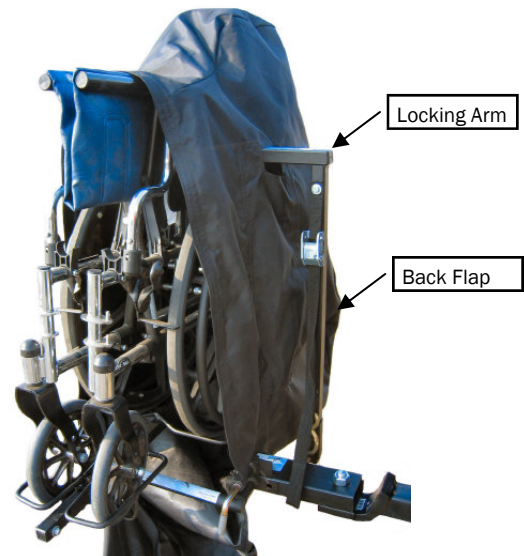

3. Bring Small Flap of cover up and around front wheels and over arm rests. Press Velcro closed.

4. Cover installation completed.

Happy Traveling!



**** Be sure Locking Arm is connected
before driving**

WheelChair Carrier

203 Matzinger Road
Toledo, OH 43612

P.O. Box 70
Lambertville, MI 48144

Phone: 419-478-4423
Fax: 419-476-1324

Website: www.WheelChairCarrier.com

Email: Admin@WheelChairCarrier.com
